16B016A SUSPENSORY MALE URINAL contains Latex rubber

The materials used in the manufacture of this urinal are virtually free from "odor cling" and are easily cleaned. To cleanse, just wash with soap and water and dry with a cloth. Urinals may be aired by placing them out of doors, but care should be taken that they are not exposed to heat or direct sunlight, both of which tend to cause the rubber to become dry and hard and lose its elasticity. Do not place in clothes dryer.

FOR BEST RESULTS: Two urinals should be used so that one may be worn while the other is cleansed and allowed to dry thoroughly. Some salves and ointments contain ingredients which are injurious to rubber. These should be avoided, but if contact is a necessity the rubber should be immediately cleansed with soap and water.

DIRECTIONS FOR FITTING:

Fitting the sheath: Place forefinger inside of urinal sheath and reverse the inner part of the sheath. This sheath is tapered to allow adjustments in size. Place the first two fingers of both hands inside of the sheath and stretch it apart. Insert penis inside as far as possible. If inner sheath is too snug, remove from penis and cut 1/8" (0.32 cm) from the end with a large pair of scissors. Repeat the above operations until sheath fits snugly but comfortably. Note: The inner sheath must fit snugly to prevent leakage. To be assured of this, do not cut more than 1/8" (0.32 cm) from the sheath at a time.

Fitting of the jock: At this point, the jock portion should be placed on the body and adjusted to size. A snug fit should be obtained. DO NOT WEAR WAIST BELT TOO TIGHTLY. If one of the smaller sizes (adjustments) is used, the excess may be cut off the waist band at the back with a pair of scissors. After having determined the proper fit of the jock, remove the jock and snap the rubber sheath into position. Remove the outlet closure cap from the bottom portion of the sheath and screw the leg bag into position on the bottom, with or without connector tubing attached.

Fitting of leg bag: Pass the thigh straps between the legs and around the thighs and snap onto the leg bag. Before adjusting the leg straps on the leg bag, be sure straps are under the leg bag and not over it. If too tight or too loose, leg straps may be adjusted and excess cut off. To keep frayed ends of cut straps from collecting dirt, apply a coat or two of clear nail polish. Bottom valve should be closed snugly to prevent leakage.

Cleaning: The rubber sheath portion, the jock and the leg bag should be washed separately.

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