
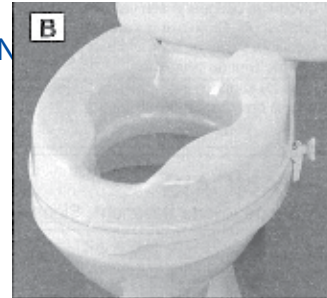
16B025 & 16B027 PARSONS ADL RAISED TOILET SEATS – 2 in & 4 in (5 & 10 cm)

Intended uses: Toilet aids allow for, or increase, independence when using the toilet, particularly by compensation for impaired movement.

Instructions for installation:

1. Raise the toilet seat that is attached to the toilet. The Parsons ADL Raised Toilet Seat (RTS) will be put onto the toilet bowl.
2. Place the Parsons ADL Toilet Seat upside down on a flat surface and slip one of the L-shaped brackets into one of the slots in the side of the seat (see figure A). Push lightly on the bracket and turn the knob clockwise until the thread engages. Repeat with the second bracket.
3. Turn the knobs until the brackets are farther apart than the width of the toilet bowl.
4. Turn the seat over and place it on the toilet bowl with the narrow cutout at the front and the brackets near the back. Position the seat with the front lip on the seat against the outside front of the bowl.
5. Tighten both brackets until they are snug against the bowl. **DO NOT OVER TIGHTEN**
6. Please note: The Parsons ADL raised toilet seat is not compatible with all toilets on the market. A properly positioned and fitted Parsons ADL Raised Toilet Seat is shown in Figure B.
7. Regular checks of the brackets should be made, ensuring that the toilet seat is secure. Spare brackets are available if required. The product code for replacement brackets is 16B026 for a set of two.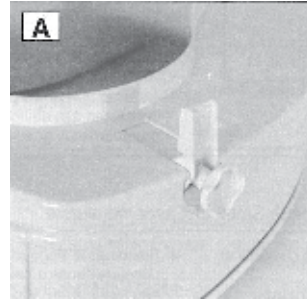

Care and cleaning

All parts can be cleaned with soap and hot water (up to 85°C). The adjusting screws can be popped out of the brackets for cleaning.

Cautions!

- The toilet seat is not to be used to support more than 190kg /420 lb.
- Do not hold onto the seat as a support when getting on and off the seat
- Be sure the toilet seat is securely in place before putting weight on it
- Before each use, please ensure there is full contact between the rubber pad on the bracket and the toilet bowl
- Under no circumstances should you stand on the raised toilet seat

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