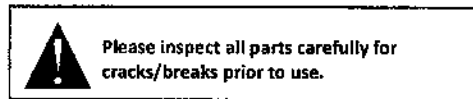
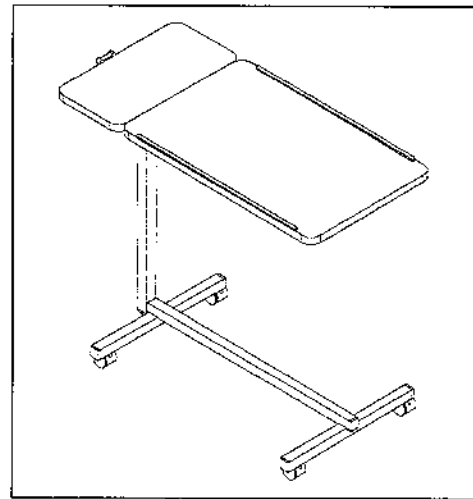
## Tilt-Top Overbed Table 20H001

### USER INSTRUCTIONS

The Twin-Top Table comes with two tops with easy height adjustment. The fixed top allows user to keep their drink while the tilt top allows users to turn to their desired reading angle. The table top height can be easily adjusted by releasing the height adjustment knob.

**!** Do not lean on the suspended end of table top at any time. This may tip over the entire overbed table. Do not put cigarettes on the table top as it may catch fire. Make sure the tilt control knob and height control knob are tightened at all times. Otherwise items on top of the table may tip over.

### FEATURES

1. Top height can be easily adjusted from 700mm to 1100mm.
2. One fixed top 200x384x20mm and one tilt top 600x384x20mm. Both are made of particle board.
3. Four easy rolling ø40mm casters provide smooth and stable mobility.
4. Tilt function allows user to turn the table to preferred angle for reading.

**!** Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance.

Parsons ADL Inc.  
1986 Sideroad 15, Tottenham, ON L0G 1W0  
(905) 936-3580 / (800) 263-1281

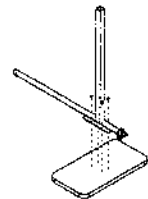
[www.parsonsadl.com](http://www.parsonsadl.com)

### ASSEMBLY INSTRUCTIONS


1. Screw in four ø40mm casters onto the two base legs.

2. Assemble two base legs to the outer mast frame.

3. Assemble the inner mast (with tilt mechanism) to the fixed top.

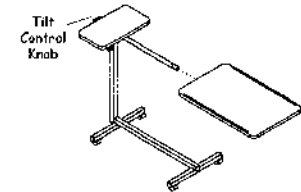4. Insert the inner mast with tilt mechanism and fixed top into the outer mast frame. Then tighten the height adjustment knob.



6. Table top heights can be adjusted from 700mm to 1100mm by releasing the height adjustment knob. Tilt top can be turned to desired angle by releasing the tilt control knob.

### MAINTENANCE INSTRUCTIONS

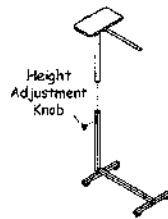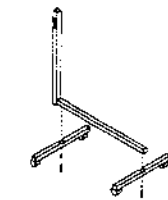1. Periodically check fit and tightness of all nuts, bolts, and adjustment knobs to make certain table is safe for use. Gently wipe the overbed table with a damp, soft cloth to remove dust and dirt.

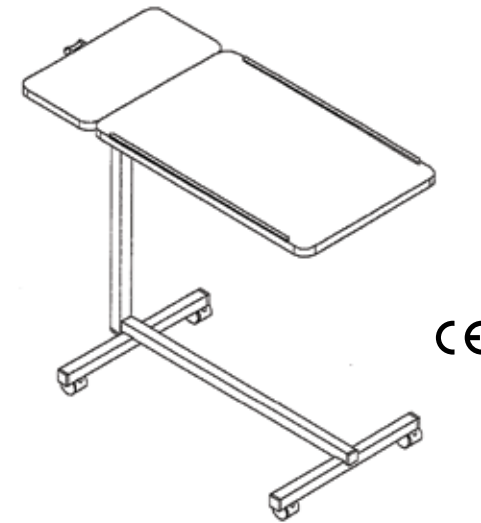
2. To wash off more stubborn grime, clean with mild soap and water. Do not use cleansers that may damage the finish. Be certain to thoroughly dry overbed table with a soft cloth.

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